


BENUTZERINFORMATIONEN

ultraSAVER Rettungstuch

SAN-0159 (ohne Fußsack)

SAN-0555 (mit Fußsack)

Maße: 200 x 70 cm

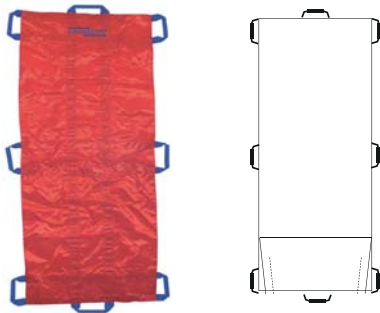

250 kg
Belastbarkeit
Load capacity



Das Rettungstuch kann auf verschiedene Weise benutzt werden.

Bei einer Nutzung mit 2 Rettern, müssen auf beiden Seiten jeweils die obersten und die untersten Griffe benutzt werden.

Bei mehr als 2 Rettern, achten Sie darauf, dass eine gleichmäßige Spannung am Rettungstuch aufgebaut wird, um den Verletzten zu transportieren.

Grundsätzlich sollten die Retter körperlich in der Verfassung sein, den Verletzten zu tragen. Es sollte auch gewährleistet werden, dass die Retter mit der Handhabung des Rettungstuchs vertraut sind.

Instandhaltung

Das Rettungstuch sollte nach jedem Einsatz und mind. 1 x jährlich auf schadhafte Stellen oder Beschädigung geprüft werden. Im Falle einer Beschädigung muss das Rettungstuch sofort außer Betrieb genommen werden.

Reinigungs- und Pflegehinweise

- Desinfektion gem. Empfehlung RKI „Flächendesinfektion nach Patientenkontakt“, bzw. mit handelsüblichen Desinfektionsmitteln.




USER INFORMATION

ultraSAVER Rescue mat

SAN-0159 (without foot rest pocket)

SAN-0555 (with foot rest pocket)

Dimensions: 200 x 70 cm

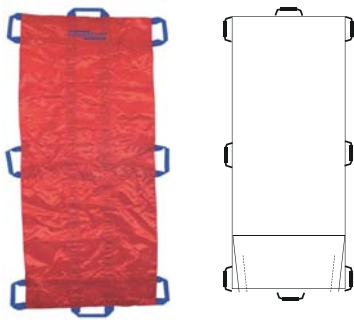

250 kg
Belastbarkeit
Load capacity



The rescue mat can be used in different ways.

If it is used by 2 rescuers, the uppermost and lowermost handles on both sides must be used.

If there are more than 2 rescuers, ensure that the rescue mat is tensioned equally for transporting the injured person.

As a general rule, the rescuers should have the physical capacity to lift and carry an injured person. In addition, they should be familiar with the use of the transfer sheet.

Maintenance

The rescue mat should be checked for signs of wear or damage after every use and at least once per year. In the event of damage or suspected damages, remove the product from operation immediately.

Cleaning and care

Desinfektion in accordance with the recommendations of the Robert Koch Institute „Surface disinfection after contact with patients“ or with a commercially available disinfectant.

