Rotaid AED Cabinets
SOLID PLUS


INSTALLATIEHANDLEIDING • GUIDE D'INSTALLATION • MONTAGEANLEITUNG
INSTALLATION MANUAL • MANUALE DI INSTALAZIONE
MANUAL DE INSTRUCCIONES • INSTRUKCJA MONTAŻU

Parts-list

This Rotaid package contains:

- Installation manual
- Polycarbonate backplate
- Polycarbonate cover
- Alarm (if ordered)
- Fastening materials

*Screw Set:

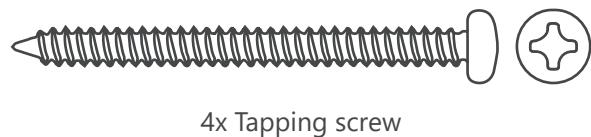Index

pag. 4		English instructions
pag. 5		Nederlandse instructies
pag. 6		Instructions en Français
pag. 7		Deutsche Anleitung
pag. 8		Instrucciones en Español
pag. 9		Istruzioni Italiano
pag. 10		Polskie instrukcje
pag. 11		User manual

Rotaid contact information

Keizersveld 52D
5803 AN
Venray
The Netherlands

www.rotaid.com

info@rotaid.com

+31 (0)478 769 052


ROTAID SOLID PLUS Installation manual

1. Determine the location where you wish to mount the Rotaid. Make sure there are no irregularities present at the desired location. Please take the fire safety and evacuation regulations into account. If mounted on a stone or concrete wall, please use the wall anchors prior to fixation.

2. Mark the 4 holes for drilling and use the screws and rings for fixation.

3. Use an Allen key (5mm) to firmly attach the nylon belt to the back plate.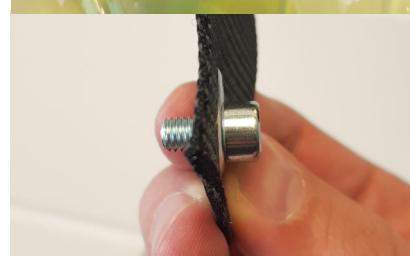

4. After attaching the cover and belt, now please use both long bolts to find the perfect position for your AED. Please make sure the bolts are firmly screwed into the block.

5. You can now place the AED into the cabinet by using its handle or carrying case handle. Please rotate the cover on the back plate by starting slightly counterwise and rotate to the right until the cover reaches the stop. When correctly placed, the hook on the bottom of the cabinet falls through the cover.

Your Rotaid Plus is now ready for use!

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