ultraMEDIC[®]

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Content

	Page
1. Product description	2
2. Important information	8
2.1 Intended use of the product	8
2.2 User groups	8
2.3 Warranty terms	8
2.4 Declaration of conformity	8
3. Information on the safety and protection of the product	9
4. Use of the ultraHEAD-LOCK III	9
5. Cleaning and Care	10
6. Maintenance	10
7. Storage	11
8. Service life	11
9. Optional accessories	11
10. Identification and Warranty Certificate	13

2. Important information

Before using the ultraHEAD-LOCK III, please read the instructions for use and safety instructions carefully. Keep the instructions for use in a safe place to avoid operating errors.

Should you require further information or encounter difficulties in its use or maintenance, please contact your local dealership or ultraMEDIC GmbH directly. Always specify the LOT- or serial number to identify the ultraHEAD-LOCK III.

2.1 Intended use of the product

The ultraHEAD-LOCK III is a medical device is used as a head restraint system for ultraCOMBI STRETCHER II (SAN-0298).

2.2 User groups

This product may only be used by qualified, expert personnel. A prerequisite is theoretical and practical experience in handling head restraint systems.

2.3 Warranty terms

The manufacturer grants a warranty period of 24 months on this product. The warranty only encompasses the costs of the spare parts.

The following points are not covered by the warranty:

- possibly omitted shipping or labour costs incurred on the purchaser's side;
- defects in the product due to normal wear and tear;
- damages caused by the user;
- defects or damages caused by non-compliance with the instructions in this document;
- any modifications or repairs to the product without prior approval by ultraMEDIC GmbH.

2.4. Declaration of conformity

The ultraHEAD-LOCK III was manufactured in accordance with the current EU regulation 2017/745. A declaration of conformity for these products can be acquired from the local dealership or the manufacturer.

3. Information on the safety and protection of the product

1. Before and after every use, check the components and patient restraining belts for damage and completeness.
2. Ensure that the patient restraining belts are correctly installed.
3. Before initial use, conduct a test run with the ultraHEAD-LOCK III to become aware of the handling of the product.
4. In the event of damage or suspected damage to the straps or the ultraHEAD-LOCK III, remove the product from operation immediately until any occurring irregularities have been remedied.
5. All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

4. Use of the ultraHEAD-LOCK III

Manual immobilisation may not be interrupted when applying the ultraHEAD-LOCK III on the patient.

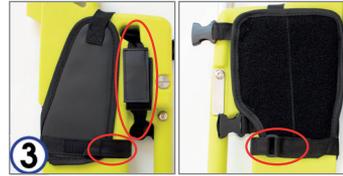
1. Setting up the head immobilizer, starting by the left part.
2. Identify the correct position of the two parts of the ultraHEAD-LOCK III thanks to the side of plastic buckles: side buckles are to be positioned on the outer side (fig. 1).

3. Insert the left part of the ultraHEAD-LOCK III from the top of the left part the ultraCOMBI STRETCHER II (fig. 2).

-
- Then, fasten the two straps to fix the ultraHEAD LOCK III to the ultraCOMBI STRETCHER II using Velcro on both straps (fig. 3).


- Repeat the operation 3 and 4 with the right side of the ultraHEAD LOCK III.

- Cushions can now be applied on each side of the head immobilizer. The flat side of each cushion has to be positioned toward the interior of the head immobilizer, where the head of the victim will be placed. (fig. 4).

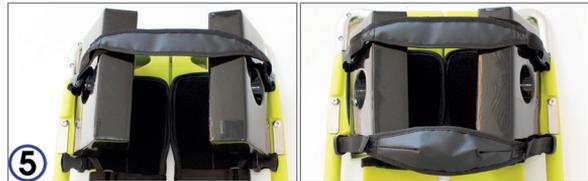- Finally, fasten and adjust the forehead strap on each side. Repeat the same operation with the chin strap. (fig. 5).

5. Cleaning and Care

Never use products with aggressive agents (bleach, ammonia, etc.) for cleaning, avoid the use of scratching materials to remove stains such as brushes, steel wool and knife blades.

For **disinfection**, you can use commercially available alcohol-based agents. Other solutions or cleaning chemicals may damage the material.

Straps: The straps that have gotten wet should only dry naturally. Avoid direct heat radiation from heat sources. These factors could damage the material.

Plastic parts: Clean with a sponge drenched with water diluted with disinfectant and then rinse and dry completely.

6. Maintenance

In order to ensure the safety of the user and the injured party during a rescue mission, we recommend that you have the ultraHEAD-LOCK III and straps visually inspected by an expert on an **annual** basis.

For German users, we recommend consideration to the following regulations and guidelines of MPBetreibV.

For users outside of Germany, a regular check according to country-specific requirements is required.

The following parts must be checked and the following defects must be excluded:

Base plate	deformation, tears, breaks, broken parts
Straps	Cutting and abrasion points, damaged seams, tissue-damaging stains
Plastic parts	Deformation, tears, breaks
other damages	

In the event of damage or suspected damages, remove the product from operation immediately until any occurring irregularities have been remedied.

7. Storage

Store the ultraHEAD-LOCK III in a dry and dark place. Avoid direct sunlight and direct heat radiation from heat sources. These factors can damage the material.

Protect the blocking devices and moving parts from dust and dirt deposits to maintain their maximum effectiveness during use.

8. Service life

The service life of the ultraHEAD-LOCK III depends on the individual conditions of use. Depending on how frequently and intensely the ultraHEAD-LOCK III is used, wear and tear on the product is normal. In this regard, correct use is assumed. The material is subject to an ageing process even if handled carefully. The product should be discarded depending on its condition.

9. Optional accessories

SAN-0291-P Base plate of ultraHEAD-LOCK III (PVC, with large Velcro area, black) and a set of forehead and chin straps

SAN-0291-B Set of two black cushions with Velcro

ultraMEDIC®

Impressum:

Corben SARL

59 Rue Stendhal
76620 Le Havre
France

Tel.: +33 2 35 300 185

Fax: +33 2 35 300 186

E-Mail: corben@corben.fr

ultraMEDIC GmbH

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Tel.: +49 2631 96983 - 0

Fax: +49 2631 96983 - 33

E-Mail: info@ultramedic.de

Web: www.ultramedic.de