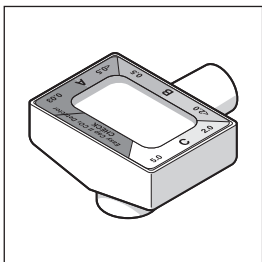
# EasyCap® II

CO<sub>2</sub> Detector

English	2
Français	6
Deutsch	10
Nederlands	14
Español	18
Italiano	22
Português	26
Svenska	30

# EasyCap® II

CO<sub>2</sub> Detector

 Rx ONLY

## Directions for Use

### Description

The Easy Cap® II detector, when connected between an endotracheal tube and a breathing device, detects approximate ranges of end-tidal CO<sub>2</sub> by color comparison in patients weighing more than 15 kg (33 lb). The detector may be used during patient transport or in any location where intubations are performed. It may be used for up to 2 hours.

### Indications

Use to assist verification of tube placement during endotracheal or nasotracheal intubation.

Use on intubated patients to detect approximate ranges of end-tidal CO<sub>2</sub> when clinically significant.

### Contraindications

- Do not use to detect hypercarbia.
- Do not use to detect main stem bronchial intubation.
- Do not use during mouth-to-tube ventilation.
- Do not use to detect oropharyngeal tube placement. Standard clinical assessment must be used.

### WARNING:

1. Before using the Easy Cap II detector, read complete Directions for Use.
2. The Easy Cap II detector is not suitable for interpretation by individuals with blue-yellow color blindness.
3. Do not use this device on patients with body weight less than 15 kg (33 lb) due to the potential for rebreathing exhaled CO<sub>2</sub>.

### Instructions for Use

1. Remove detector from pouch and immediately match the initial color of the indicator to the purple color stripe labeled CHECK around the detector window as shown in **A**.  
Use the detector only if the purple color of the indicator, is the same color or darker than the stripe marked CHECK.
2. Insert the endotracheal tube. Inflate the cuff, if equipped.
3. Firmly attach the Easy Cap II detector to the endotracheal tube; then attach the breathing device, as shown in **B**.

4. When a Heat and Moisture Exchanger (HME) is used, note:
  - Connect the HME to the endotracheal tube
  - Connect the Easy Cap II detector to the HME
  - Connect the breathing device to the Easy Cap II detector
5. Ventilate the patient with SIX BREATHS of moderate tidal volume (may be done slowly or quickly). Compare the indicator color in the window on full-end expiration to ranges printed on the detector cover. Interpret results according to the decision tree that follows.  
If Easy Cap II detector results are not conclusive, the endotracheal tube should be immediately reinserted unless correct anatomic location can be confirmed with certainty by other means.

### WARNING: INTERPRETING RESULTS BEFORE CONFIRMING SIX BREATH CYCLES CAN YIELD FALSE RESULTS.

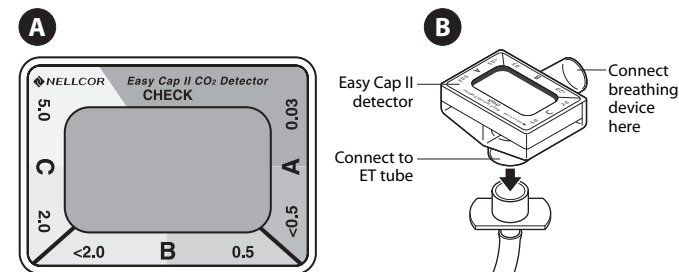
Gastric distention with air prior to attempted intubation may introduce CO<sub>2</sub> levels as high as 4.5% into the Easy Cap II detector if the endotracheal tube is misplaced in the esophagus. Initial Easy Cap II detector color (yellow) may be interpreted as a false positive if read before delivery of six breaths.

### WARNING: THE EASY CAP II DETECTOR WILL RESPOND TO CO<sub>2</sub> IN THE PRESENCE OF A MAIN STEM BRONCHIAL INTUBATION. STANDARD CLINICAL ASSESSMENT SHOULD BE USED TO CONFIRM PROPER POSITION OF THE ENDOTRACHEAL TUBE WITHIN THE TRACHEA.

6. Note that the indicator color will continue to fluctuate from inspiration to expiration for up to 2 hours. During this period, due to varying patient humidity, lighting, or distance from the device, the color change on inspiration may not be as apparent as initially observed. However, the end-tidal CO<sub>2</sub> range can still be read on **full end-expiration**.
7. Remove and discard the device after use, **not to exceed 2 hours**.

### CAUTION

 STORE AT OR BELOW 24°C (75°F). STORAGE FOR EXTENDED PERIODS AT TEMPERATURES ABOVE 24°C (75°F) MAY REDUCE SHELF LIFE.

## CAUTIONS:

1. Inspect the Easy Cap II detector before attaching it to the endotracheal tube. Do not use if any physical damage is observed.
2. Do not use device in the presence of trichlorethylene or chloroform anesthetic; chemical interactions will affect device accuracy.
3. Do not subject device to extreme humidity, such as a heated humidifier or nebulizer; excessive humidity will affect device accuracy.
4. Reflux of gastric contents, mucus, edema fluid, or intratracheal epinephrine into the Easy Cap II detector can yield persistent patchy yellow or white discoloration which *does not vary with the respiratory cycle*. Contamination of this type may also increase airway resistance and affect ventilation. Discard device if this occurs.
5. In cardiac arrest, re-establishment of cardiac output and pulmonary blood flow by adequate CPR is necessary to increase end-tidal CO<sub>2</sub> to levels detectable by the Easy Cap II detector (above 0.5%).
6. The Easy Cap II detector is not a substitute for observation of the patient. This device must not be relied upon as the sole indicator of resuscitation performance.

## Mechanical Specifications

Internal Volume:	25 cc
Resistance to Flow:	3.0 cm H <sub>2</sub> O ± 1 cm at 60 L/min flow
Weight:	Less than 20 g
Connector Ports:	Patient end: 15 mm I.D. Circuit end: 15 mm O.D.

Covered by one or more of the following U.S. Patents and foreign equivalents: 4,728,499; 4,879,999; 5,166,075; and 5,179,002.

© 2005 Nellcor Puritan Bennett Incorporated. All rights reserved.

## Interpretation

### Color Range A

End-Tidal CO<sub>2</sub> level approximately 0.03% to < 0.5%. < 4 mmHg, < 0.5 kPa

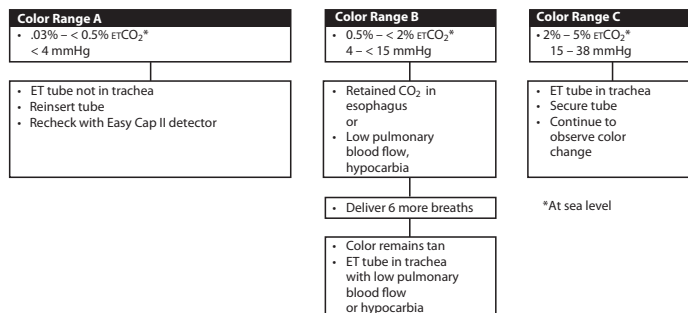
### Color Range B

End-Tidal CO<sub>2</sub> level approximately 0.5% to < 2%. 4 to < 15 mmHg, 0.5–2 kPa


### Color Range C

End-Tidal CO<sub>2</sub> level approximately 2% to 5%. 15 to 38 mmHg, 2–5 kPa

## Easy Cap II Interpretation in the Presence of Adequate Pulmonary Blood Flow/Spontaneous Heartbeat:
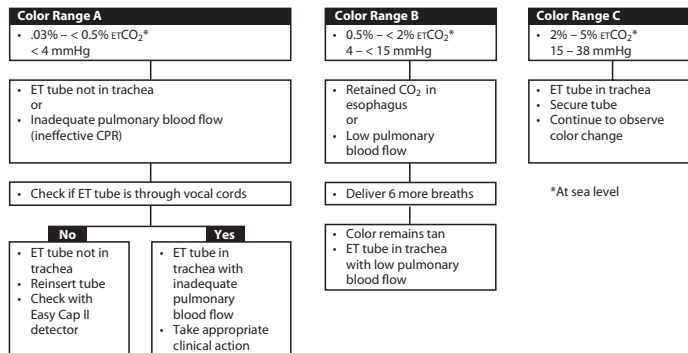
## Interpretation in the Presence of Poor Pulmonary Blood Flow/Cardiac Arrest:

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© 2005 Nellcor Puritan Bennett Inc. Alla rättigheter förbehålles.
Tyco Healthcare UK LTD